

Quick guide Bonalive® putty MIS


**Please note the following when using the product.
Please also read the instructions for use.**

Non-hardening and ready-to-use paste for filling bony voids and gaps. No mixing, moistening or other preparation is needed prior to application.


Can be used to extend autologous bone graft. Excess fluid/blood needs to be removed from the autograft before mixing with BonAlive® putty. The binder may start to dissolve prematurely in contact with fluid for an extended period.

Use of antibiotics, and other treatments involved in the implantation procedure should accord with routine clinical practice.


A thorough surgical debridement must be performed and the surface of the bone needs to be refreshed before implantation.

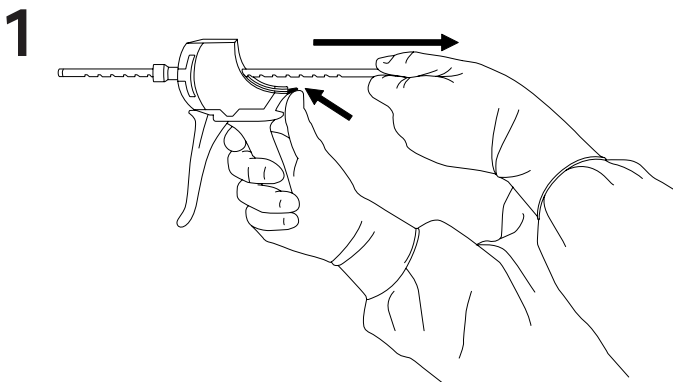


Release the dispenser plunger by lifting the black release latch and pull the plunger to its rearmost position.

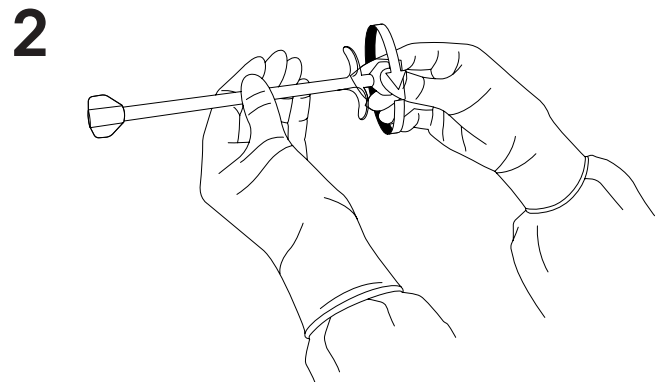

Unscrew the cap from the cartridge end that has "wings".

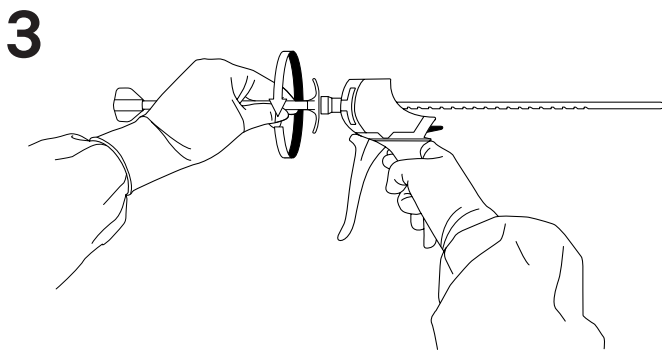

Attach the cartridge by screwing it firmly onto the dispenser. Unscrew the second cap from the cartridge.

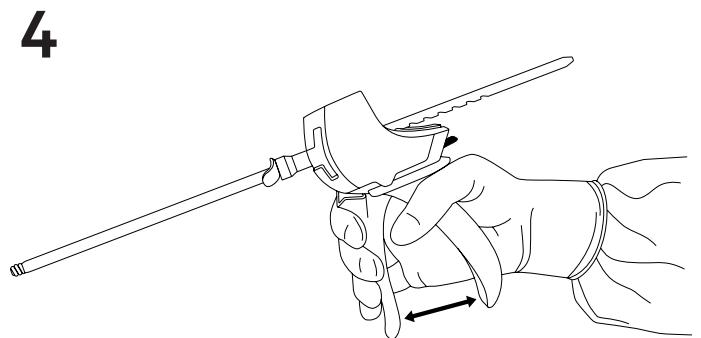

Gently press the dispenser trigger repeatedly to fill the defect. Each press gives approximately 0.25 cc.