


Please note the following when using the product.
Please also read the instructions for use.

Non-hardening and ready-to-use paste for filling bony voids and gaps. No mixing, moistening or other preparation is needed prior to application.


Can be used to extend autologous bone graft. Excess fluid/blood needs to be removed from the autograft before mixing with BonAlive® putty. Mixing with autograft is preferable done just before implantation. The binder may start to dissolve prematurely in contact with fluid for an extended period.

Use of antibiotics, and other treatments involved in the implantation procedure should accord with routine clinical practice.


A thorough surgical debridement must be performed and the surface of the bone needs to be refreshed before implantation.

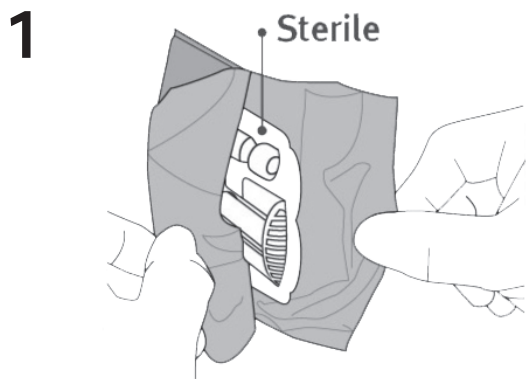

Peel open the pouch and aseptically remove the sterile tray.
Detach the applicator from the tray.

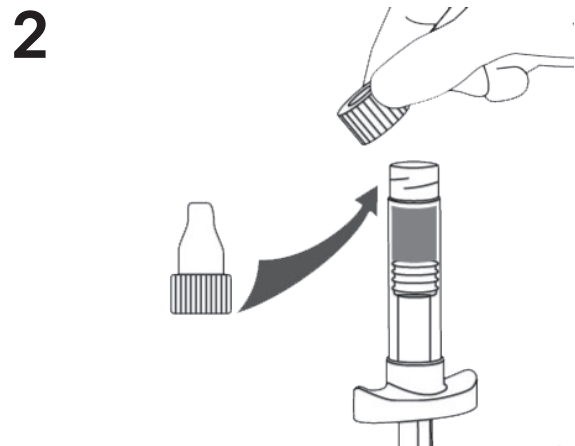

Unscrew the cap. Screw tightly the nozzle onto the applicator body. Alternatively, without the nozzle, push the plunger rod to force the putty to a sterile cup and subsequently perform the implantation with a sterile instrument.

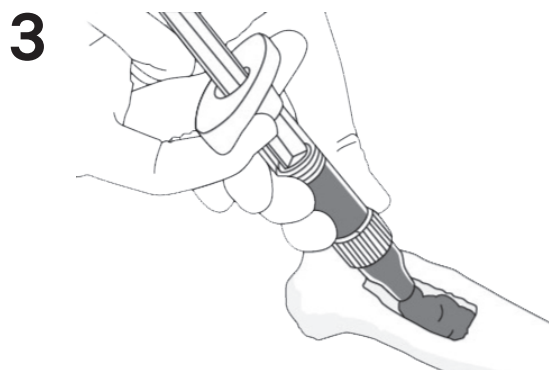

Push the plunger rod to force the putty into the nozzle.
Move the applicator to the defect site.
Push the plunger rod to gently fill the defect with the putty.